





## Full Face Mask

### FITTING YOUR MASK

- 

1 Ensure that both Headgear Clips (**D**) are unhooked from the Frame (**B**).
- 

2 Place the Seal (**A**) onto your face and guide the Headgear (**C**) over your head.
- 

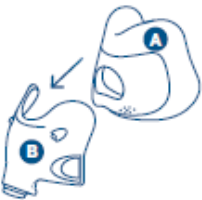
3 Hook both Headgear Clips (**D**) onto the Frame (**B**).
- 

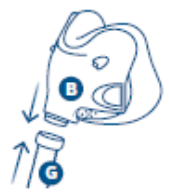
4 Gently tighten the Headgear (**C**) straps, starting with the blue upper straps, followed by loosely fitting the lower straps.
- 5 Connect the Tube (**G**) with the Swivel (**E**) to the PAP tube. Turn on the PAP device. Repeat step 4 until the mask seals and the Seal (**A**) feels comfortable on your face. If you are experiencing mask leaks, you may need to lift the Seal (**A**) off your face and reposition it. If the Headgear (**C**) touches the ear, adjust the straps on the top of your head.

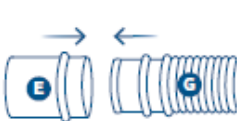


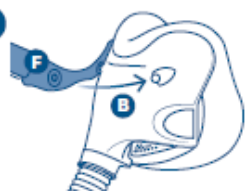
The F&P myMask™ App is designed to support patient mask setup. F&P myMask helps you and your patients effectively fit, fine-tune and clean F&P CPAP\* masks for the treatment of obstructive sleep apnea.

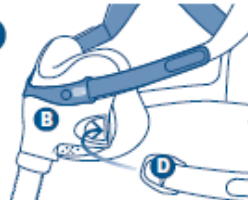
### ASSEMBLING YOUR MASK

- 

1 Push the Seal (**A**) firmly into place on the Frame (**B**) to click into place.
- 

2 Push the Tube (**G**) onto the Frame (**B**) to click into place.
- 

3 Push the Swivel (**E**) onto the Tube (**G**) to click into place.
- 

4 Connect the Frame Clip (**F**) to the Frame (**B**) to click into place.
- 

5 Hook the Headgear Clips (**D**) onto the Frame (**B**) to click into place.